

RELAX**SAN**[®]
M E D I C A L E



The healthy way for a beautiful life



I N D E X

RELAXAN® MEDICALE

Page 04 General Presentation

Page 05 Compression Classes

Page 06 RelaxSan® Medicale Classic

Page 07 RelaxSan® Medicale Soft

Page 08 RelaxSan® Medicale Silver

Page 09 RelaxSan® Medicale Cotton

Page 10 RelaxSan® Medicale Anti-Embolism

Page 11 Size Chart

Page 12 How to wear the
RelaxSan® Medicale stockings

Page 13 Advices and contraindications

Page 14 Address and Company's Information

RELAXAN® MEDICALE

Therapeutic stockings from GT Calze srl

RelaxSan® MedicaLe is a complete line of high quality therapeutic graduated compression stockings and pantie hoses resulting from a long research on new technologies and production materials. With this line of high quality products, GT Calze srl offers an important contribution to the treatment and prevention of venous and circulatory diseases.

RelaxSan® MedicaLe line of stockings is organized into 5 product categories:

RelaxSan® MedicaLe Classic

RelaxSan® MedicaLe Soft


RelaxSan® MedicaLe Silver

RelaxSan® MedicaLe Cotton

RelaxSan® MedicaLe Anti-Embolism

Our wide range of products guarantees an important and complete support for different lifestyles and periods of year.

Detailed information about RelaxSan® MedicaLe product line on next pages.

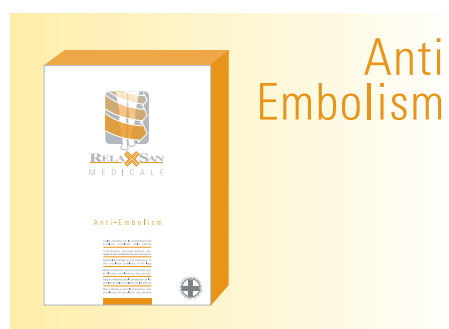

COMPRESSION CLASSES

of **RelaxSan® Medicale** line

The therapeutic graduated compression stockings are subdivided into compression Classes according to the experimental UNI ENV 12718 European standard depending on the compression strength at the ankles (point b).

Ccl A – 10-14 mmHg – light compression

Ccl I – 15-21 mmHg – medium compression

Ccl II – 23-32 mmHg – moderate compression

Ccl III – 34-46 mmHg – strong compression

Ccl IV - >49 mmHg – extra-strong compression

The therapeutic graduated compression stockings are normally prescribed by specialized physicians who recommend the most suitable garment and compression for the pathology to be treated. The 3 different compression classes of the therapeutic RelaxSan® Medicale stockings are generally prescribed for the following venous and lower extremity disease:

Ccl I - 15-21 mmHg - Medium Compression

– Indicated for the prophylaxis of venous diseases in risk patients, tired and heavy sensation in the legs, appearance of varicose veins during pregnancy, thrombosis and embolism.

Ccl II - 23-32 mmHg - Moderate Compression


– For slight chronic venous insufficiency, thrombophlebitis (superficial thrombosis) with and without edema, post-sclerotherapy, after selective out-patient treatment of varices and stripping, varicose veins during pregnancy, prophylaxis of thrombosis and embolism.

Ccl III - 34-46 mmHg – Strong Compression

– Specially designed for advanced phases of chronic venous insufficiency, after recovery from crural ulcer (prophylaxis of relapse), posttraumatic edema, and reversible lymphedemas.

The RelaxSan® Medicale therapeutic stockings are manufactured according to the European standards for Medical Devices and EC certified.

Compression

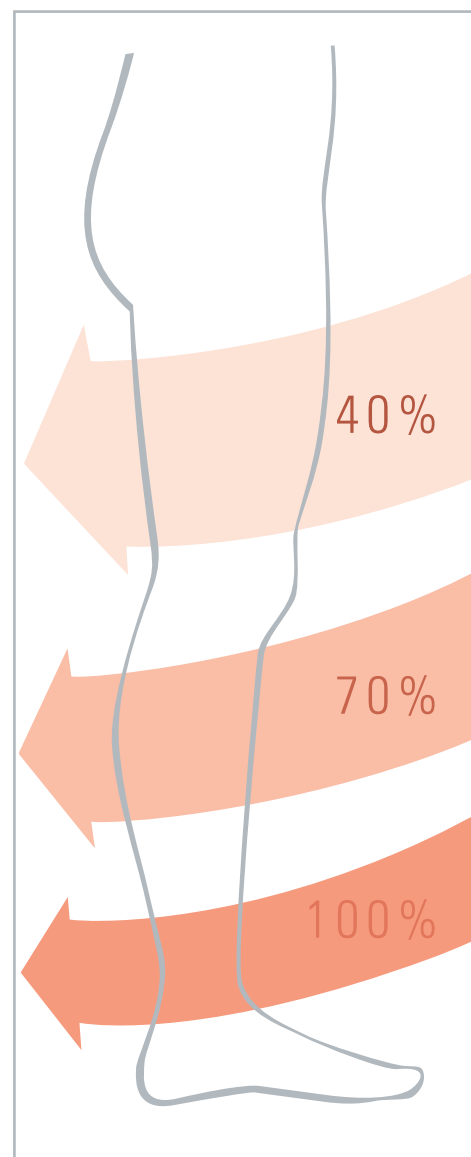

RELAXAN® MEDICALE

Classic

Classic line of therapeutic graduated compression stockings manufactured using the highest quality yarns. A line of most resistant and effective articles with refined knitting and accuracy of details. These stockings are produced in Ccl. I–II–III compression Classes.

Class I

material composition:

75% Poliammidica/Nylon/Polyamid/Polyamide (PA)
25% Elastan/Elastane/Elasthan/Elasthanne (EA)

Class II

material composition:

70% Poliammidica/Nylon/Polyamid/Polyamide (PA)
30% Elastan/Elastane/Elasthan/Elasthanne (EA)







Class III

material composition:

70% Poliammidica/Nylon/Polyamid/Polyamide (PA)
30% Elastan/Elastane/Elasthan/Elasthanne (EA)



Articles available in the CLASSIC line

Classic	AD	AG	AGH	AGTL-AGTR	AT	ATM	
							
Class I	OTCTCN	OTCTCN	OTCTCN	OTCTCN	OTCTCN	OTCTCN	
Class II	OTCTCN	OTCTCN	OTCTCN	OTCTCN	OTCTCN	OTCTCN	
Class III	OTCTCN	OTCTCN	OTCTCN		OTCTCN		
Legend	OT Open Toe		CT Closed Toe		C Short sizes		N Standard sizes
Colours	BEIGE						

RELAXAN® MEDICALE

Soft

This new line is manufactured using advanced technologies and soft microfiber fabrics for most comfortable and soft products. Excellent finish quality. These stockings are produced in Ccl. I–II compression Classes.

Class I

material composition:

75% Poliammidica/Nylon/Polyamid/Polyamide (PA)

25% Elastan/Elastane/Elasthan/Elasthanne (EA)

Class II





material composition:

73% Poliammidica/Nylon/Polyamid/Polyamide (PA)

27% Elastan/Elastane/Elasthan/Elasthanne (EA)



Articles available in the SOFT line

Soft	AD	AGH	AT	ATM
				
Class I	OT CT N	OT CT N	OT CT N	OT CT N
Class II	OT CT N	OT CT N	OT CT N	OT CT N
Legend	OT Open Toe CT Closed Toe C Short sizes N Standard sizes			
Colours	BEIGE - BLACK			

RELAXAN® MEDICALE

Silver


Therapeutic stockings produced with X-Static® silver plated yarn. Thanks to the countless properties of silver, these garments are antistatic, antibacterial, thermoregulator, anti-odor, and therapeutic. These garments of excellent quality and properties are produced in the Ccl II compression Class.

Material composition:

72% Poliammidica/Nylon/Polyamid/Polyamide (PA)

18% Elastan/Elastane/Elasthan/Elasthanne (EA)

10% Silver Fiber X-Static

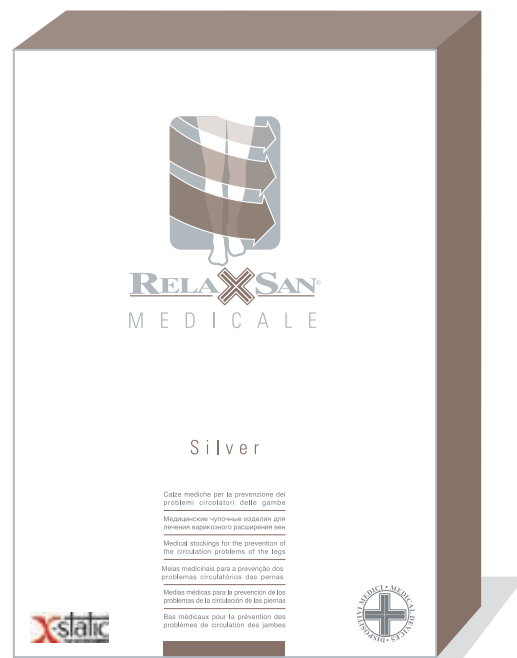

Articles available in the SILVER line

Silver	AD
Class II	OT N
Legend	OT Open Toe CT Closed Toe C Short sizes N Standard sizes
Colours	BEIGE BLACK



NATURAL

100%. It's safe, non toxic, does not contain any chemical or pesticides residues.



ANTISTATIC

For its high electric conductivity.



THERMO-REGULATOR

Fresh in summer and warm in winter for its thermal conductivity; both the reflecting and the radiating power improve humidity passage.



THERAPEUTIC

Thanks to its conductive characteristics this new fibre gives some benefits to health and improves blood circulation.



ANTIMICROBIAL

Removes those bacteria which causes smell and removes fungi responsible for the "athlete's foot".



ODOUR CONTROL

Neutralizes ammonia, denatured proteins and blocks bacterial proliferation.

RELAXAN® MEDICALE

Cotton

Latest line of graduated compression therapeutic stockings made of cotton, with a natural feel against the skin and extremely soft thanks to the high quality cotton yarns used for its manufacturing. This line is produced in the Ccl I-II compression Classes.

Class I

material composition:

53% Cotone/Cotton/Baumwolle/Coton (CO)

29% Elastan/Elastane/Elasthan/Elasthanne (EA)

18% Poliammidica/Nylon/Polyamid/Polyamide (PA)

Class II

material composition:

51% Cotone/Cotton/Baumwolle/Coton (CO)

31% Elastan/Elastane/Elasthan/Elasthanne (EA)

18% Poliammidica/Nylon/Polyamid/Polyamide (PA)



Articles available in the COTTON line

Cotton	AD	AGH	AT	ATM
Class I	OT CT N	OT CT N	OT CT N	OT CT N
Class II	OT CT N	OT CT N	OT CT N	OT CT N
Legend	OT Open Toe CT Closed Toe C Short sizes N Standard sizes			
Colours	BEIGE - BLACK			

RELAXAN® MEDICALE

Anti-Embolism

Line of graduated compression products specifically designed to prevent thromboembolic problems.

Material composition




Material composition:

70% Poliammidica/Nylon/Polyamid/Polyamide (PA)

30% Elastan/Elastane/Elasthan/Elasthanne (EA)



Articles available in the ANTI-EMBOLISM line

Anti-Embolism	AD	AGH	AGTL-AGTR
			
18-21 mmHg	OT N	OT N	OT N
Legend	OT Open Toe CT Closed Toe C Short sizes N Standard sizes		
Colours	WHITE		

SIZE Chart

The choice of the right size is extremely important for the correct functionality of the therapeutic stockings.

To determine and chose the right size, measure the girth size of the legs on the cB, cC, cD, cF and cG points as shown in the picture. In any case, we recommend to refer to your doctor for the right size and leg girth measurement.




Size Chart

Girth sizes in cm					
SIZES	S	M	L	XL	XXL
N C	1	2	3	4	5
cG	48-56	52-60	56-64	60-68	64-72
cF	39-44	43-48	47-52	51-56	55-60
cD	26-31	30-35	34-39	38-43	42-47
cC	29-34	32-37	35-40	38-43	42-47
cB	19-21	21-23	23-25	25-27	27-29

Length sizes in cm					
SIZES	S	M	L	XL	XXL
N	1	2	3	4	5
IG	71-72	72-73	73-74	74-75	75-76
IF	59-60	60-61	61-62	62-63	63-64
ID	38-39	39-40	39-40	40-41	40-41
IC	30-31	31-32	31-32	32-33	32-33
IB	11	11	12	12	12

Length sizes in cm exclusively for the SHORT sizes of the RelaxSan® Medicales CLASSIC line					
SIZES	S	M	L	XL	XXL
C	1	2	3	4	5
IG	65-66	66-67	66-67	67-68	67-68
IF	54-55	55-56	55-56	56-57	56-57
ID	35-36	35-36	36-37	36-37	36-37
IC	27-28	27-28	28-29	28-29	28-29
IB	10	10	11	11	11

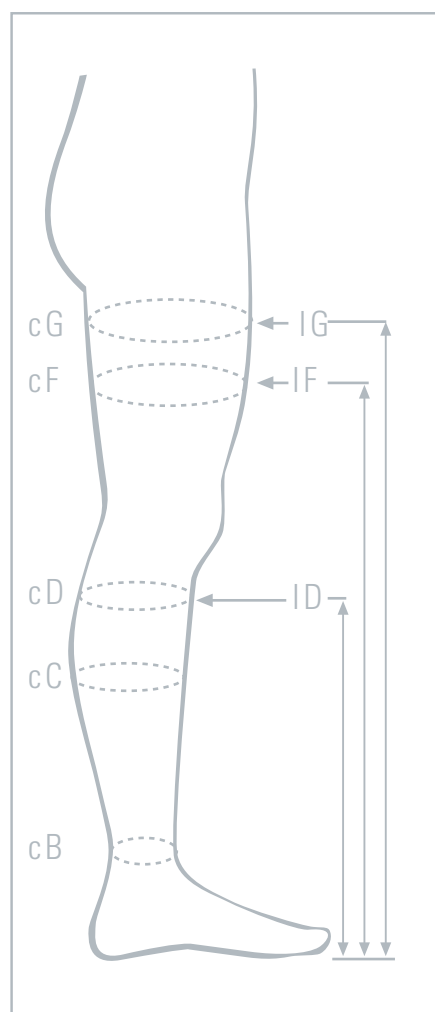

HOW TO WEAR

the RelaxSan® Medicales stockings

The correct wearing of the graduated compression stockings makes this difficult operation easier and guarantees the perfect functionality and comfort of the garment.

1 Roll down the stockings to the heel gently.

2 Insert the foot into the foot of the stocking. Adjust the stockings at the heel.

3 Unroll the stockings smoothing them up to the knee with the palms. DO NOT pull the stockings by the rubber band on the open end.

Repeat the same operation on the other leg wearing the garment in linear position, avoiding twisting.

For Hold-up stockings and Pantie hoses

pull up the stockings to the groin smoothing them uniformly over the legs.

IMPORTANT – do not wear the hold-up stockings pulling them by the silicone band so as to avoid garment damages and allow a uniform distribution over the leg. For Pantie hoses – smooth and adjust the upper part of the garment so that the front and rear seams are perfectly vertical.



Hold-up stockings



Pantie hoses

ADVISES

We advise you to wear the RelaxSan® Medicales stockings in the morning, before getting up, when the legs are still fresh and not yet swollen. Before wearing the stockings, you should take off rings and pieces of jewellery so as to avoid damaging the garment.




CONTRAINDICATIONS

This treatment does not present secondary effects if you follow the size chart supplied by the manufacturer carefully. Some contraindications, however, should be considered and observed:

1 We advise you against the use of these stockings during the night's rest, as the horizontal position of the body already helps the blood circulation.



2 These stockings are not recommended for subjects suffering from dermatosis with secretion.

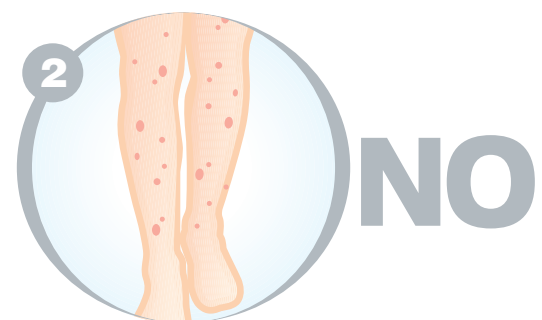

3 These stockings are not recommended for subjects suffering from allergy to the material used in their manufacturing.



The manufacturer will not be held responsible for any consequences caused by the non-observance of the above-mentioned contraindications.

THE COMPANY

Calze G.T. s.r.l.

Via Walter Tobagi 17/19/21

46040 Casaloldo Mantova – Italy


Tel. +39 (0)376 780686 – 779834

Fax +39 (0)376 781843

e-mail info@relaxsan.it

web site www.relaxsan.it



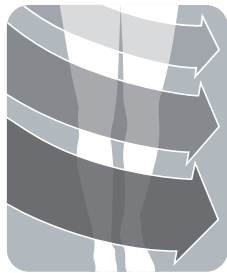

RELAXSAN[®]
M E D I C A L E

Distributed in

From.....